MS Schippers

One-stop-shop for livestock farmers

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EN MS GluePro

Characteristics

- Fast curing polyurethane adhesive glue, especially developed to mount wooden or rubber blocks underneath bovine feet.
- Easy to apply.
- Simple one-step process mixing according to the correct ratio of both components is done in the mixing tip.
- Glue can be applied with a regular dosing gun on either the claw or the hoof block.
- 7 to 9 applications with one cartridge.
- Unique “easy snap off top” to open the cartridge easily without any tools or pliers.
- Neutral scent

Use

- For best results: store and use MS GluePro at room temperature (15- 25°C).
- Avoid long term exposure to extremely high or low temperatures.
- If the product is too warm or too cold, it should be brought to correct core temperature slowly.
- Always use a mixing tip and a well-fitted dosing/application gun.
- Hold the cartridge in a vertical position and pump out a small quantity of adhesive. This way possible bubbles will be removed and the glue will always be mixed according to the correct ratio. Ensure that both components are free of air bubbles before the mixing tip is placed on the cartridge.
- Do not allow glue to squeeze out between hoof and block. For best results leave a layer of 3 mm between hoof block and claw as a solid foundation and for an optimal fixation. Limit movement of the hoof block to a minimum after application.
- Leave the mixing tip on the cartridge during storage to maximize the storage period.
- Make sure both openings are completely free of polluted or hardened product residues before placing a new mixing tip on the cartridge. Especially when the cartridge will not be used for a longer period. When in doubt, use a (disposable) needle to open de cartridge.

Application

1. Remove the “easy snap off top” easily by pressing with a thumb.
2. Place the MS GluePro cartridge in the dosing gun.
3. Hold the cartridge in a vertical position and pump out a small amount of adhesive, so that both components flow simultaneously.
4. Place the mixing tip.
5. Clean the claw thoroughly.
6. Dry the claw with blow dryer or paint stripper.
7. Apply MS GluePro on hoof block or directly on the claw.
8. Place the hoof block and hold firmly without movement for several seconds.
9. For best results make sure an adhesive layer of 3 mm thickness is applied.
10. Let the glue set for approx. 3 minutes before releasing the animal.

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© Copyright 2014