MS Semenstorage 130L

4508427

NL	MS Semenstorage 130L	
	Manual.	2
EN	MS Semenstorage 130L	
	Manual.	10
DE	MS Spermaschrank 130L	
	Manual.	15
FR	MS Conservateur de semence 130L	
	Manual.	21
ES	MS Nevera para conservación de semen 130L	
	Manual.	27
IT	MS Stoccaggio Seme 130L	
	Manual.	33

MS Semenstorage 130L

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 Replacing the regulator unit..... 14

MS Semen Storage

The 130-litre MS Semen Storage is especially designed for storing diluted semen at 17 °C. It has built-in heating, cooling and air circulation, which keeps the internal temperature constant at 17 °C, in all ambient temperatures between + 8 °C and + 30 °C.

No technical changes to the installation are permitted, unless the manufacturer has given express written permission. Without the abovementioned permission, the manufacturer's warranty becomes void. This guide contains information regarding operation, maintenance and tracing and troubleshooting faults. You should take note of all relevant information in this manual before installing or operating the system. This is important for averting material damage and personal injury. Careful and precise assembly results in good operation.

Conditions

MS Schippers assumes no responsibility for errors contained herein or for any direct or indirect damages or consequential damages of any nature whatsoever arising from or in connection with provision, function or use of this document.

MS Schippers reserves the right at any time, without prior notice, to make changes to the structure of equipment or the information in the manual, without any resulting obligation to apply these changes to products already delivered.

In case of misuse, the manufacturer's guarantee and responsibility becomes void.

Recommendations

- Always transport the MS Semen Storage vertically.
- Avoid shocks during transport.
- When the MS Semen Storage is not in use, turn it off, this will save electricity.
- Do not place the MS Semen Storage in direct sunlight, because that will lead to unnecessary energy consumption.
- Do not place the MS Semen Storage in animal housing, because of the corrosive atmosphere.
- Do not overfill the MS Semen Storage unit – The bags should lie flat on the trays.
- Do not place any products against the rear of the MS Semen Storage unit *good air circulation is essential*.
- There are no user serviceable parts. In case of problems, always *contact customer service at MS Schippers*.

Installing the MS Semen Storage unit:

- Place the MS Semen Storage unit on a flat, dry, horizontal surface in a dust and ammonia free environment (e.g. in the office or canteen).
- At least 150 mm free space should be allowed on either side of the MS Semen Storage unit, to allow air circulation.
- Do not turn on the unit for the first 24 hours following transport!
- Plug the MS Semen Storage unit into a standard 230 V 13 A mains socket.
- Depending on the ambient temperature, the MS Semen Storage unit will take a certain amount of time to reach the set temperature.

Specifications:

The MS Semen Storage is fitted with both cooling and heating systems and built-in fans to maintain an even internal temperature.

The temperature is set to 17 °C as standard + / - 0.5 °C.

Dimensions Packed

Height	885 mm
Width	570 mm
Depth	600 mm
Weight	36.5 kg

Dimensions Product

Height	845 mm
Width	545 mm
Depth	570 mm
Weight	34 kg

CFC/HFC-free	Yes
Refrigerant type	R600a

Voltage	230 V AC
Current	0.52 A
Consumption	120 W

Net cooling space capacity	130 litres
Number of trays	4 full-depth

Environmental temperature – range	+ 8 °C /+30 °C
Target temperature	17 °C +/- 0.5 °C

Sound level	41 dB
-------------	-------

Service manual

Routine maintenance

The heat condensers are mounted in the sides of the MS Semen Storage.

- a) Check that there is sufficient space and airflow either side to prevent a build-up of heat. This will compromise performance in high ambient temperatures.

The compressor is located in the base of the MS Semen Storage.

- b) Check that there is no debris or build-up of fluff and dust. This will compromise performance in high ambient temperatures.
- c) Check that the internal-condensation drainage hole is clear.
- d) Check the door seals for damage/wear. If you detect any gaps, you should replace the seal immediately.
- e) Check the evaporation tray mounted above the compressor this should be clean and dry.
- f) Check the condition of the cable and three-pole plug. Replace both if there is any sign of damage.

The shelves are coated with low-density white polythene, to prevent damage to the semen bags.

- g) Check each shelf for damage and if the coating is missing or worn then replace the shelf.

The MS Semen Storage unit has one internal circulation fan, fitted in the back of the cabinet.

- h) Check that the fan is spinning and that there is no build-up of dust or fluff.

If the fans in the heating unit are blocked then follow the procedure in section 3.

Replacing the regulator unit

The main regulation unit is located in the inside of the cabinet.

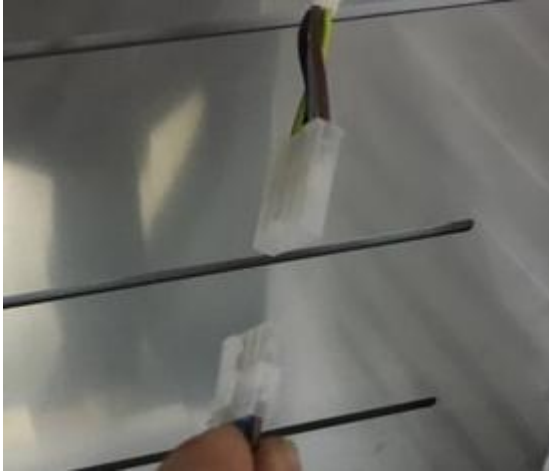
All important components are mounted on that unit. If there is any defect, the cabinet can be repaired by replacing that unit.

You replace the regulation unit as follows:

Plug & Play Storage Cabinet 4508427

1. Take the unit's plug out of the power outlet
2. Remove the three grills form the cabinet
3. Unscrew the outer four screws (see image) Attention! do not remove the screws holding the fan in place

4. Remove the unit carefully (wiggle it a little)
5. Remove the plug-in connector – squeeze it inwards and it will release
6. Pack the return-for-exchange unit safely for transport
7. Unpack the new unit
8. Connect it to the unit's connector
9. Place the unit carefully into the housing Attention! Make sure that the cables are not trapped anywhere
10. Re-insert the four screws and tighten them
11. Replace the three grills in the cabinet
12. Close the door and plug the unit into the mains
13. The storage cabinet will start automatically

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**Schippers Italia Srl**
**Schippers Schweiz GmbH**
**Schippers UK Ltd.**
**LLC Schippers Russia**